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**HK Pregnancy Tester  
for Sheeps and Goats  
Operating Instructions**

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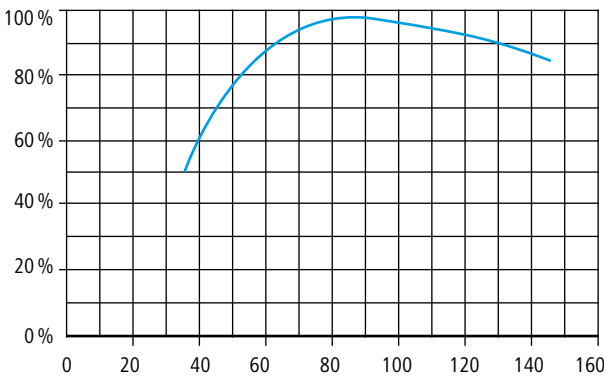
The probe should be placed in the the stifle fold in front of and above the udder on the abdominal wall and directed to the animals opposite side, between the hip-bone and the rear rip.

**Anteile der richtigen Diagnose** • pourcentage des diagnostics justes • percentage of correct diagnoses

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Results of a pregnancy diagnosis test from 167 goats and 122 sheep between the 35th and 120th day after mating (Reference: DLG-report no. 4621)

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# HK pregnancy diagnosis unit for sheep and goats

With growth of the foetus and increase in the amniotic fluid, the uterus considerably enlarges as pregnancy progresses. In the 8th to 9th week of pregnancy, the uterus will lower into the abdominal cavity and its amniotic fluid is measurable by the ultrasonic waves. Pregnancy and readiness for operation are indicated audibly (signal sound) and visually (LED). An existing pregnancy is indicated by a permanent signal. An interrupted signal indicates adequate contact between the probe and the skin surface. The HK pregnancy tester is recommended from the 60th day after mating in goats and from the 70th day after mating in sheep with a high diagnosis-precision.

**Important notice:** This appliance is supplied with a 9 volt alkaline-manganese battery. A reusable battery with battery charger can be ordered as an accessory. Please only use original parts for this appliance and follow the operating instructions. Under no circumstances should the appliance be operated with a battery charger if the battery has been inserted, since this can result in severe damage. To obtain a reusable battery and battery charger, please contact your specialist supplier or the manufacturer direct at [www.rheintechnik.com](http://www.rheintechnik.com)

## 1. Functional test

Switch on the appliance by operating the ON/OFF key. Moisten the probe head with a few drops of contact oil (see details under 2). Then press the enclosed test piece on the probe. If a continuous signal sounds or appears, the device is ready for operation. You must always check the battery status lamp. If the red lamp lights up, the battery must be changed immediately. No further tests must be carried out in this case, since the power supply required for a reliable test result is no longer assured.

## 2. Contact oil

The contact oil ensures airless contact between the animal's skin and the face of the probe, which is necessary for an accurate diagnosis. Before each diagnosis, apply a liberal amount of oil on the face of the probe. With the exception of mineral oils, all clean oils are suitable for this purpose. Paraffin oil, for example, has good properties. Contact gels are also suitable and are often preferred as hands and test unit stay cleaner.

## 3. Diagnosis spot and scanning-direction of the probe

The optimal diagnosis spot is situated in the stifle fold in front of and above the udder. The probe is lightly pressed against the abdominal wall and directed upwards to the animal's opposite side, between the hip bone and the rear hip. When putting the rear leg a little backward, it is easier to reach the spot. By moving the probe slightly forward and backward, the ultrasonic waves scan the medium part of the abdominal region. In the case of the unit with integrated ultrasonic head, the whole unit has to be directed accordingly.

## 4. Diagnosis

If a continuous signal of at least 1-2 seconds sounds or appears, the animal is pregnant. A non-continuous tone indicates a sufficient contact between the probe and the skin; the animal is not pregnant.

## 5. Diagnosis precision

Diagnosis precision is optimal from the 60th to 120th day after mating when working with goats, and from the 75th and 120th day after mating when working with sheep. However, the following aspects should be taken into consideration:

- The animal to be tested should stand in a natural position.
- If the probe is moved too much backwards, towards the tail, the ultrasonic waves can be reflected by the fluid-filled bladder giving a false positive result. This risk can be minimised by removing access to drinking water for a couple of hours before testing.
- Apply fresh oil or gel on the face of the probe for each new placing.
- In case of negative result, repeat testing at a later date. This gives additional certainty.
- Gain some experience of testing by using animals which are known to be pregnant or not pregnant respectively.

## 6. Automatic switch-off

The appliance switches off automatically after 3 minutes if no contact has been registered on the ultrasonic head within this time. To restart, press the ON/OFF switch.

## 7. Changing the battery

To change the battery, unscrew the two screws on the battery compartment at the back of the appliance using a Phillips screwdriver. Take off the lid, remove the used battery from the clip and insert the new battery. Close the battery compartment. You must make absolutely sure that the lid is properly fitted. Caution: avoid squeezing the cable. Please only use 9 volt alkaline-manganese batteries. Other types of battery impair the test performance and serviceability.

## 8. Looking after the appliance

The appliance should be thoroughly cleaned with a damp, clean cloth after each use. Persistent residues, particularly on the probe rubber of the unit with an external probe, cause damage and reduce the serviceable life. Do not throw the appliance or let it drop, since this could damage the probe and the electronics. We recommend that you store the appliance in its case.

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## Warranty

Based on our General Terms and Conditions of Business, we offer a 2 years warranty for this device covering damage caused by material or manufacturing defects. This warranty begins on the date of purchase. The warranty does not cover:

1. Damaged caused by natural wear and tear (wear parts, sliding parts), excessive strain and incorrect use (non-compliance with operating instructions).
2. Damage caused by force majeure or sustained during transport.
3. Damage to rubber parts, e.g. on connecting lines. Warranty repairs are only carried out by the manufacturing plant or an authorised contracted workshop.

The manufacturer reserves the right to reject any warranty claims in the event of non-compliance. The device must for this purpose be returned to the point of sale, at the risk of the device owner, complete, packaged and with sufficient postage, including the original invoice (address see dealer stamp). Please provide as much information as possible on the nature of the problem. Costs incurred as a result of unjustified defect claims will be invoiced to the device's owner.

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## CE-conformity declaration

Albert KERBL GmbH hereby declares that the product / device described in these instructions complies with the fundamental requirements and other relevant stipulations and regulations. The CE mark confirms compliance with the Directives of the European Union. The declaration of conformity can be viewed by visiting [www.kerbl.com](http://www.kerbl.com)

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## Electronic scrap

Disposing of this device after its service life is the responsibility of the operator. Please consult the valid national regulations. The device must not be disposed of in household waste. In accordance with the stipulations of the EU Directive on the Disposal of Electrical and Electronic Devices, the device can be disposed of free of charge at the local waste collection or recycling centre. Alternatively, it can be returned to retailers who offer a collection service. The proper disposal helps to ensure environmental protection and prevents any adverse effects on human health and the environment.

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## Albert Kerbl GmbH